


# QuickProfile™ 2019-nCoV IgG/IgM Combo Test Card

POCT CE Made in the USA



- ✓ Quick results in 15 minutes
- ✓ Whole Blood, Serum, and Plasma Sample
- ✓ Easy to Use
- ✓ Anti-viral IgG/IgM Test

## One - Step Rapid Test for COVID-19 Diagnosis

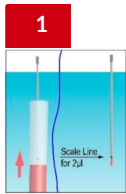


Collect and prepare the sample: Whole blood, serum, or plasma.



Using the capillary pipet, transfer 2ul of sample to the area marked as "S1".




Add 2 drops of sample buffer (80-100 ul) to well marked as "S"

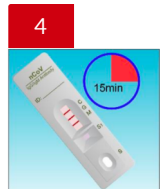

Read the result at 15 minutes. A strong positive sample may show result earlier.

### Results Interpretation

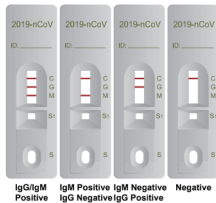

IgG/IgM Positive    IgM Positive IgG Negative    IgM Negative IgG Positive    Negative



Invalid

### Clinical Studies

TEST RESULTS	POSITIVE	NEGATIVE
POSITIVE SAMPLE	65	3
NEGATIVE SAMPLE	9	302
TOTAL SAMPLES	74	305
ACCURACY	<b>87.8%</b>	<b>99.0%</b>

IgM and IgG combined specificity and sensitivity is 99.0% and 87.8%, respectively.

Total accuracy equals 96.8%

### Ordering Information

Catalog No.	Description	Packaging
71108B	QuickProfile™ 2019-nCoV IgG/IgM Test Card	25 tests/ kit

Lumiquick Diagnostics, Inc.

2946 Scott Blvd, Santa Clara, CA 95054, US

Tel: +1(408)855-0061 - Toll Free USA: 1(888)352-8394 - Fax: +1(408)855-0063 - info@lumiquick.com